

THE BREWER COMPANY™

Alleviate™

SUPPORT SYSTEM

Installation & Operation Manual



THE BREWER COMPANY™

more possibilities

13901 Main Street • Menomonee Falls, WI 53051
phone: 1.888.BREWER.1 (1.888.273.9371)
web: www.brewercompany.com
fax: 1.262.251.1786


Service

If you require assistance with the installation or operation of your Brewer Alleivate Support System, call the Brewer Customer Service Department at **1-800-558-8777**. Our trained staff will help you.

Assembly

Insert support plate into the bracket located under the seat. Mount the support plate to the center hole location using the star washer, washer and knob supplied.

Position support plate at the desired width then tighten knob securely.

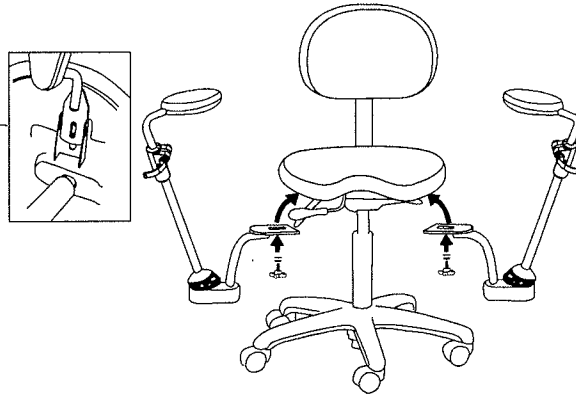


Elbow Support Height Adjustment

The elbow height can be adjusted to accommodate the user's optimal work position.

Turn the latch outward and adjust the elbow support up or down to the desired height, then push the latch fully inward to lock in place.

NOTE: The latch mechanism tension is preset at the factory but may loosen with use. To set proper tension, fully open the latch and turn mounting hardware finger tight plus 1/2 turn. Do not overtighten. Close latch and test to ensure latch and elbow support is secure.

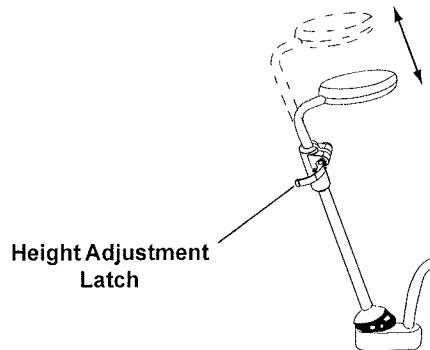


Free Movement Adjustment

The tension of the elbow support can be adjusted to increase or decrease ease of movement.


Turn the adjustment knob clockwise to tighten the ball and socket; turn counterclockwise to loosen.

NOTE: Even when tightened securely, there will be some slight movement in the elbow support assembly. This is an intended feature of the elbow support system.

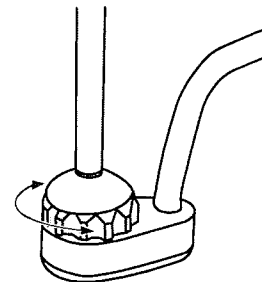

Operation

Before sitting down, always make sure to push elbow supports away from the chair; either to the side or back.

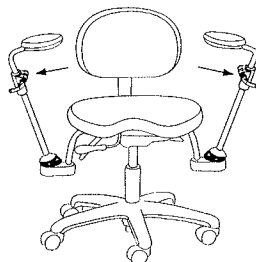

WARNING

Do not use the elbow supports to get in or out of the chair, or injury to the user or damage to the supports could occur.

